


AMO ENDOSOL

BALANCED SALT SOLUTION FOR OPHTHALMIC IRRIGATION (REF 15020)

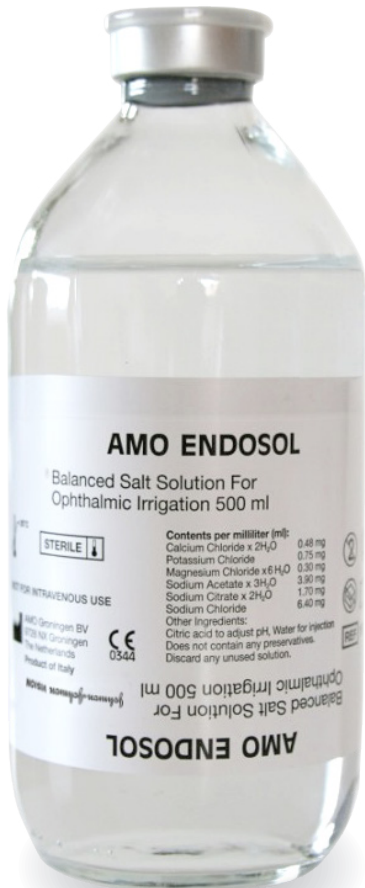

- Sterile, isotonic and without preservatives
- 500 ml glass bottle

COMPOSITION

1 ml of solution contains

Calcium Chloride x 2H₂O 0.48 mg

Potassium Chloride 0.75 mg

Magnesium Chloride x 6H₂O 0.30 mg

Sodium Acetate x 3H₂O 3.90 mg

Sodium Citrate x 2H₂O 1.70 mg

Sodium Chloride 6.40 mg

Other ingredients:
Citric acid to adjust pH

Water for injection

RANGE OF USES

For intraocular and topical irrigation of the eye during surgical procedures and for diagnostic and therapeutic measures where irrigation or retention of moisture is required, e.g.

- Longer irrigation of the anterior and posterior chambers, e.g. for cataract surgery and vitreoretinal surgery
- Surgery on the cornea
- Rehydration on the cornea during cornea transplantation
- Cooling of the cornea during photorefractive keratectomy (PRK)
- Retention of the anterior segment during trabeculectomy
- Reconstruction of the anterior segment during perfusive keratoplasty
- Enzymatic zonulolysis
- Cauterisation

AMO ENDOSOL

BALANCED SALT SOLUTION FOR OPHTHALMIC IRRIGATION (REF 15020)

INSTRUCTIONS FOR USE

Intravenous (IV) hanger is required. Remove hanger from carton and place over bottle neck. Slide hanger over the entire bottle. Invert the bottle with the hanger while simultaneously pulling the side tabs. An infusion device with air inlet drip chamber should be used as the bottle does not have an additional airway (follow the manufacturer's instructions for the particular infusion device used). Use the bottle as follows (aseptic conditions):

1. Remove the cap and disinfect the rubber stopper with an alcohol swab.
2. Insert the spike of the infusion device through the target area on the rubber stopper.

The fluid should be flowing before use to remove air from the tube.


STORAGE

- Do not store above 25°C
- Do not use after the expiry date

PACKAGING


500 ml glass bottle/10 bottles per carton


SYMBOLS USED ON THE PACKAGE

 Consult instructions for use

 Do not reuse


 Sterilized using steam or dry heat


 Do not re-sterilize

 Do not use if package is damaged

 Catalogue number

 Batch code

 Use by [YYYY-MM] year-month

 Upper limit of temperature

 Manufacturer

This piece does not replace the manufacturer's Direction For Use. Based on ENDOSOL Balanced Salt Solution DfU Z311232 Rev. D - Apr. 2019. REF2022OTH4022. For healthcare professionals only. Please reference the Instructions for Use for a complete list of Indications and Important Safety Information and contact our specialists in case of any question.

© Johnson & Johnson Surgical Vision, Inc. 2022

PP2021CT6408

Johnson & Johnson VISION